

Purify O₃[®]

Elite Series

PORTABLE OZONE SANITIZER & DEODORIZER USER MANUAL

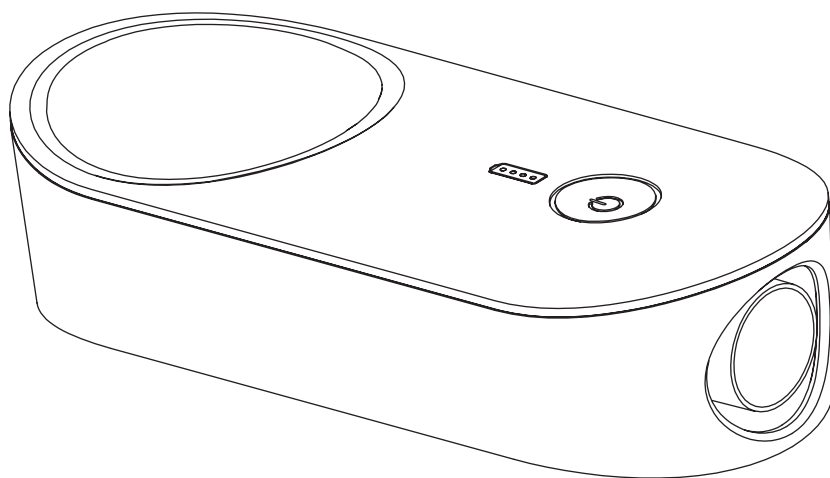


TABLE OF CONTENTS:

Charging the Battery	5
Contraindications for Use	1
Frequently Asked Questions	9
Important Notes/Warranty	4
Sanitizing Steps	6-8
Parts Included/Sanitizer Components	3
Prior to Use	4
Warnings/Symbols/Specifications	2

Thank you for purchasing Responsive Respiratory's Purify O3 Elite Sanitizer. The Purify O3 Elite utilizes ozone technology to sanitize quickly and effectively without water, soap or chemicals. With proper use, the Purify O3 Elite Sanitizer is proven to effectively kill up to 99.99% of germs and bacteria.

Purify O3 Elite is a non-sterile device, intended for single person use.








CONTRAINDICATIONS FOR USE:

Persons with underlying health conditions should consult with their health care professional prior to utilizing Purify O3 Elite to sanitize. Underlying health conditions may include, but are not limited to: lung conditions such as COPD, asthma, emphysema, cancer, as well as individuals sensitive to ozone or chemical sensitivity.





Direct exposure to ozone is not recommended, as ozone is a known airway irritant and prolonged exposure can cause damage to the lungs. Accidental inhalation may lead to minor symptoms including a mild headache, coughing, shortness of breath, irritation of skin or burning of eyes in very sensitive individuals.

Persons exposed to ozone should move to fresh air and call medical attention if necessary.

WARNINGS:

-  Read user manual carefully prior to operation for troubleshooting, maintenance and safety information. Save for future reference.
-  Do not use with liquids. This device contains electronic components.
-  Use only in a well ventilated room. Ensure Purify O3 Elite is placed in a sealed (zipped) case during the active cycle and required rest period.
-  Do not use near open flame, gas vapors or other combustible items.
-  Keep out of reach of children.
-  Do not breathe in ozone release.
-  Not for use with CPAP/BiPAP machine and accessories.

SYMBOLS:


-  This device complies with part 15 of FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.
-  Waste Electrical and Electronic Equipment (WEEE). It is the responsibility of the end user to dispose of this equipment at a designated collection point.
-  Consult Manual
-  Class 3 Electronic Appliance

SPECIFICATIONS:

Model: Purify O3 Elite Series 190-6100 (XD200)
 Size: 5.75" L x 2.5" W x 1.5" H
 Weight: 0.49 lbs
 Power Source: 3.7V, 2000mAh Rechargeable Lithium-ion battery
 Input: AC100-240V, 50/60Hz
 Rated Power: <- 15VA
 Operating Temperature: 23° F ~ 113° F / -20° C ~ 55° C
 Storage Temperature: 23° F ~ 113° F
 FDA Registered

PARTS INCLUDED:


- Purify O3 Elite sanitizer with rechargeable battery
- USB cable and charging adapter
- Dual purpose sanitizing/travel case
- User Manual

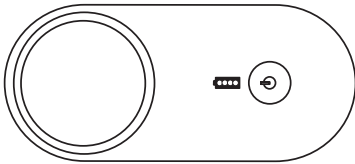


Purify O3 Elite Sanitizer



Dual Purpose Sanitizing/Travel Case
#190-6002

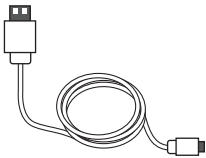


USB Cable
#190-6103

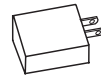

Charging Adapter
#190-6004

SANITIZER COMPONENTS:

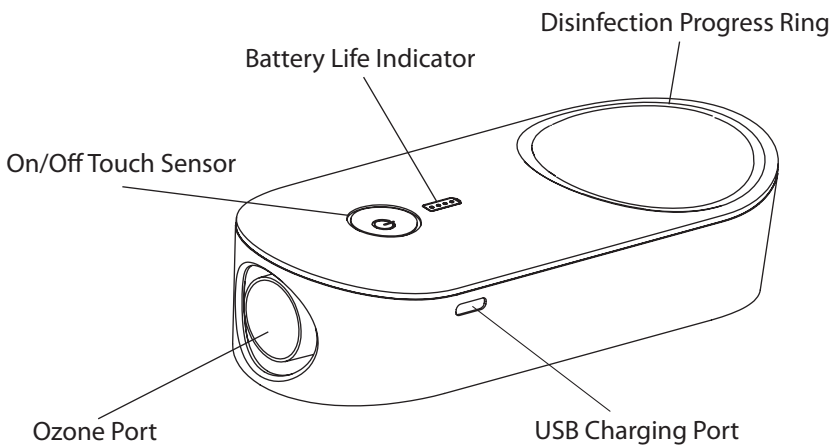

PRIOR TO USE:




Prior to use, please ensure:

Visible residue is wiped clean using a cleansing wipe or damp cloth.

Purify O3 Elite sanitizer must be used inside the sealed sanitizing/travel case.

Always use the Purify O3 Elite sanitizer in a well ventilated area. Do not breathe ozone release.

IMPORTANT NOTES:

-  For health and safety reasons, do not use open the sanitizing case for at least 90 minutes after sanitizing to allow the produced ozone to dissipate back to oxygen.
-  Ensure the Purify O3 Elite is operated in a well ventilated area. Do not breathe in ozone release during/after operation.
-  Warranty is VOID if repairs/modifications to the device are made by unauthorized persons. To obtain service within the warranty period, please contact the retailer where sanitizer was purchased.

LIMITED WARRANTY:

The Purify O3 Elite sanitizer is covered by a limited warranty for a period of two (2) years from date of purchase and covers defects in the product resulting from faults in the material or workmanship. Damage due to improper use, misuse or normal wear is not covered.

Please register your device online: www.respondo2.com/warranty and write down the date of registration in this booklet.

Warranty service and/or replacement is at the discretion of Responsive Respiratory. For warranty requests, please contact your retailer or health care dealer.


CHARGING THE BATTERY:

The Purify O3 Elite sanitizer contains a rechargeable Lithium-ion battery. For best performance and battery life, ensure the Purify O3 Elite is powered off while charging.

To charge the battery, connect the USB cable to the charging adapter. Plug the USB cable into the charging port on the side of the sanitizer as shown in Figure 1 below. Plug into an electrical outlet. During the charging process, the Battery Life Indicator will illuminate with a series of blinking white indicator lights. When the Purify O3 Elite is fully charged, the Battery Life Indicator lights will change to solid white.

Charging time averages 2-3 hours and may vary depending on the battery's remaining capacity. If the sanitizer stops working during use after a full charge, the battery may have reached the end of its serviceable life.

Figure 1

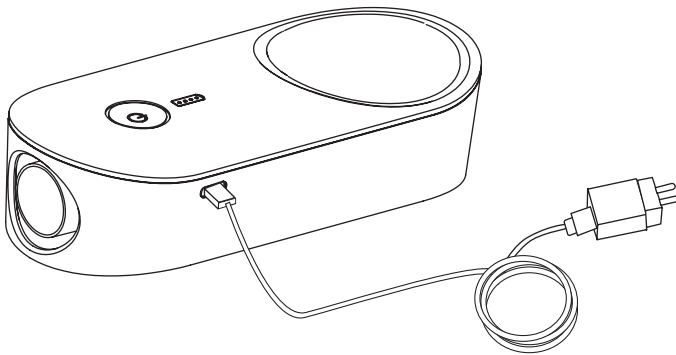

LOW POWER WARNING:


When low battery levels are reached, the Purify O3 Elite will alert the user with both visual and auditory alerts. The Battery Life Indicator lights will blink rapidly and a long buzz will sound. If in use, the sanitizer will attempt to continue sanitizing for the remainder of the available battery life. It is recommended to charge the sanitizer as soon as possible after the low power warning.

SANITIZING & DEODORIZING:

1. Wipe item to be sanitized to remove any soil or contaminants prior to sanitizing. Inspect item for signs of wear/tear, deterioration, pitting, cracks.
2. Place sanitizer and item inside the supplied sanitizing/travel case as shown in Figure 2.

WARNING:
Operate only in a well ventilated area.
Do not breathe the ozone.


Figure 2

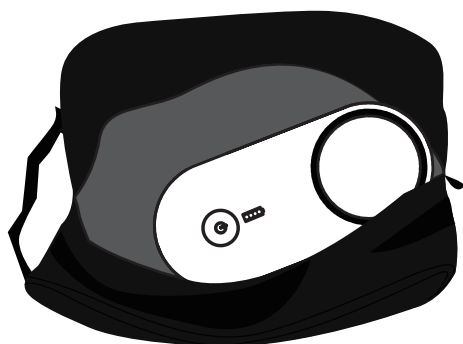

- To activate the Purify O3 Elite, place and hold finger on the On/Off Touch Sensor for 2 seconds as shown in Figure 4. The Purify O3 Elite will beep once and the Disinfection Progress Ring shown in Figure 5 will illuminate to indicate the disinfection process has begun. The Battery Life Indicator will also light to show the estimated battery life remaining.


NOTE: The On/Off Touch Sensor is activated by covering the button - it will not depress if pushed.

Figure 4

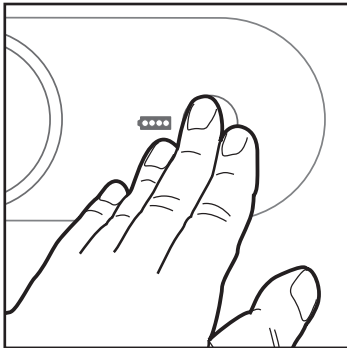

On/Off Touch Sensor


Figure 5

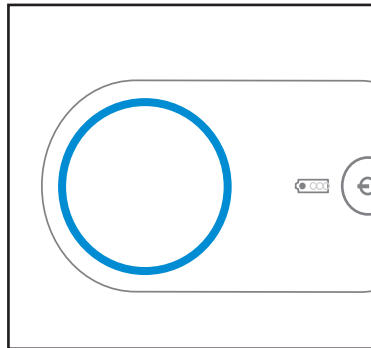


Disinfection Progress Ring

- After activating Purify O3 Elite, ozone will begin streaming. Zip case fully closed. The sanitizing/deodorizing cycle for Purify O3 Elite is approximately 35 minutes in length. During the cycle, the Progress Indicator Ring will darken to indicate when the cycle is one fourth, halfway, and three-fourths complete as shown in Figure 6. At the conclusion of disinfection, the sanitizer will beep once and power off.


Figure 6

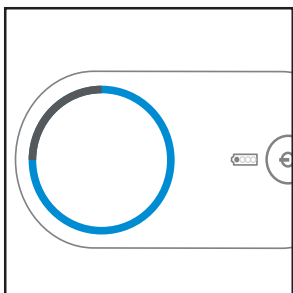

1/4 cycle

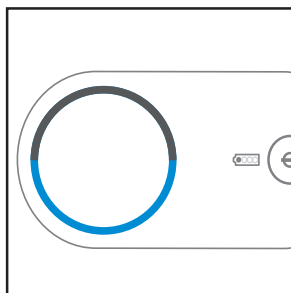

1/2 cycle

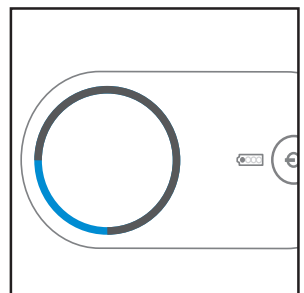

3/4 cycle

6. The sanitizer and items must remain in the closed case for an additional 90 minutes to allow the residual ozone to dissipate.

NOTE: To stop the sanitation process mid-cycle, place finger on the On/Off Touch Sensor for 4 seconds. The sanitizer will signal the cancellation with a long beep and power off. Reseal case and let sanitizer and accessories sit in closed case for at least 90 minutes to allow residual ozone to dissipate before use. Stopping a sanitizing cycle before completion is not recommended.

IMPORTANT:
Do not open case for at least 90 minutes after sanitation is complete to allow remaining ozone to dissipate.

FREQUENTLY ASKED QUESTIONS:

The disinfection process leaves an odor:

- Run the Purify O3 Elite disinfection cycle earlier in the day to allow time for the residual odor to dissipate before utilization of item.

Charging indicator light does not turn on:

- Check the charging adapter and wire connection. Ensure battery has been charged for at least one hour.

Purify O3 Elite does not turn on after charging:

- Ensure the On/Off Touch Sensor is fully covered for 2 full seconds to power on.
- Connect the Purify O3 Elite to the charger and plug into an electrical outlet. Cover the On/Off Touch Sensor for 2 seconds to power on. Disconnect from outlet and use as normal.

Battery is fully charged but Purify O3 Elite sanitizer is not working:

- Contact the retailer where sanitizer was purchased for warranty evaluation.

The Purify O3 Elite stops functioning:

- Battery may be depleted. Connect charger to sanitizer and plug into electrical outlet to charge.

Purify O3 Elite stops during use after a full charge:

- Battery may have reached the end of its serviceable life. Charge sanitizer fully and attempt to run a complete sanitizing cycle.
- If the problem persists and sanitizer is less than 2 years from date of purchase, contact the retailer the sanitizer was purchased from for warranty evaluation. If over 2 years from purchase, dispose of the product per electronic recycling guidelines in your area.

Purify O3 Elite does not power off automatically after the sanitation cycle:

- Power the sanitizer off by covering the On/Off Touch Sensor for 4 seconds.
- If the problem persists and sanitizer is less than 2 years from date of purchase, contact the retailer the sanitizer was purchased from for warranty evaluation. If over 2 years from purchase, dispose of the product per electronic recycling guidelines in your area.

No ozone is generated:

- Verify the sanitizer has adequate battery life. Connect charger to sanitizer and plug into electrical outlet. Power sanitizer on and confirm fan is working. If fan is inoperable, contact the retailer where sanitizer was purchased for warranty evaluation.

Sanitizing CPAP/BiPAP or nebulizer machines:

- Purify O3 Elite is not intended for use to sanitize CPAP equipment and accessories and should not be utilized with machines, masks, tubing and medical accessories.



CALIFORNIA PROPOSITION 65 WARNING

WARNING: This product may expose you to chemicals known to the State of California to cause cancer and birth defects or other reproductive harm.

ADVERTENCIA: Este producto puede exponerlo a sustancias químicas que el Estado de California reconoce como causantes de cáncer y defectos de nacimiento u otros daños reproductivos.

Label Statement:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

Please note that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: --Reorient or relocate the receiving antenna. --Increase the separation between the equipment and receiver. -- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. --Consult the dealer or an experienced radio/TV technician for help.



Responsive Respiratory, Inc.
6244 Lemay Ferry Road. • St. Louis, MO 63129
866-333-4030 • 866-333-4035 F
www.purifyo3.com • www.respondo2.com